

HydroCleanse™
Sinus Wash Delivery System



IRRIGATION. SUCTION.
SYNCHRONIZATION.

Medtronic

IRRIGATION INNOVATION. LESS MESS.

Help your patients find their way to sinus relief.

Sinus irrigation doesn't have to be uncomfortable and messy. The HydroCleanse™ Sinus Wash Delivery System is an office-based solution that combines pressurized sinus irrigation with built-in suction for a more comfortable patient experience. Disposable and easy to use, it allows you to irrigate the sinuses while limiting pooling and effluent discharge.



- Syringe with a self returning hand-grip that allows for consistent irrigation while minimizing the need to refill.
- Using the hand-grip, spray pressure can easily be increased or decreased.
- Syringe is filled by spiking a saline bag or using a bottle.

HydroCleanse™

Sinus Wash Delivery System



- Single hand operation
- 360° rotatable fan spray
- Suction catheter tips with three different angles: 120°, 70° and 13°
- Able to connect with an external vacuum pump for suction



The irrigator handle system includes a fan-shaped spray nozzle that provides a large spray area. The sinus specific suction catheters and single hand operation allow surgeons to access each sinus and irrigate all areas by using the rotating wheel.

Medtronic's HydroCleanse sinus wash delivery system gives you a cost effective solution that can assist with removing bacteria and debris from your patient's sinuses while they are in your office. For more information on this product contact your local Medtronic rep today.

Improve the sinus wash experience. Choose HydroCleanse.

ORDERING INFORMATION

1960100 HydroCleanse™ Sinus Wash Delivery System

PROCEDURE CODES

31000 Lavage by cannulation; maxillary sinus (antrum puncture or natural ostium)
31002 Lavage by cannulation; sphenoid sinus (antrum puncture or natural ostium)

Rx only. Refer to product instruction manual/package insert for instructions, warnings, precautions and contraindications.

For further information, please call Medtronic ENT at 800.874.5797 or consult Medtronic's website at www.medtronic.com/ent.

Medtronic

Medtronic plc ENT

6743 Southpoint Drive N
Jacksonville, FL 32216
USA
Toll free: (800) 874-5797
Telephone: (904) 296-9600
Fax: (800) 678-3995

International Telephone Numbers

Europe HQ Switzerland 41-21-802-7000
Latin America HQ 305-500-9328
Adriatic Region 385-1-488-1120
Argentina 54-11-4898-5700
Australia 1-800-668-670
Baltic Region 37-1-67560226
Belgium 32-2456-09-09
Brazil 55-11-2182-9200
Canada 1-800-268-5346
Chile 56-2-2655-5110
China 86-10-5869-8989
Colombia 57-1-742-7300
Czech Republic 420-233-059-111
France 33-470-679-800
Germany 49-2159-8149-353
Greece 30-210-67-79-099
Hong Kong 852-2919-1300
Hungary 36-188-90600

India 91-22-33074700
Israel 972-9-972-4400
Italy 39-02-24137-324
Japan 81-3-6774-4611
Korea 82-2-3404-3600
Lebanon 961-1-370-670
Luxembourg 32-2456-09-09
Malaysia 60-37-953-4800
Mexico 52-55-11-02-90-30
Netherlands 31-45-566-8800
Poland 48-22-4656900
Russian Fed. 7-495-580-73-77
Singapore 65-6436-5000
South Africa 27-11-260-9300
Spain 34-91-625-05-40
Taiwan 886-2-2183-6000
Thailand 662-232-7400
UK 44-1923-205-166